# EN Cult-Oil

## PRODUCT INFORMATION

Catalogue no.		Catalogue no.	
<b>Glass bottles</b>		<b>PETG bottles</b>	
OIL-50	50 ml	COIL-50	50 ml
OIL-100	100 ml	COIL-100	100 ml
OIL-250	250 ml	COIL-500	500 ml
OIL-500	500 ml		

### Application

IVF Basics® Cult-Oil is a ready-to-use, double washed with water, light mineral oil for use in In Vitro Fertilization (IVF), Intracytoplasmic Sperm Injection (ICSI) and related Assisted Reproductive Technologies (ART) procedures (d = 0,83-0,86 g/ml).

Light mineral oil to overlay media is highly recommended during oocyte culture, denudation of oocytes, fertilization of oocytes (either by IVF or ICSI), embryo culture and manipulation. By using IVF Basics® Cult-Oil small volumes of medium can be used for fertilization and embryo culture/handling. Primarily, a mineral oil overlay is intended to protect the culture medium from evaporation, thereby reducing harmful effects of increased osmotic pressure, avoid fluctuations in the pH and temperature. This function is critical since variations in the pH of the medium may affect embryo metabolism, whereas osmotic stress may cause DNA and protein alterations that affect cell function.  
*For professional use only.*

### Composition

Mineral Oil – Paraffin Oil (CAS code: 8042-47-5)

### Quality Control

- Endotoxin test (USP <85>): < 0,10 EU/ml by *Limulus Amebocyte Lysate methodology*
- Sterility test by the current Ph. Eur. 2.6.1/USP <71> No Growth
- Density: 0,83 – 0,86 g/ml
- Viscosity: < 60 cP at 30°C
- Peroxide Value (POV): < 0,10 mEq/kg
- Mouse Embryo Assay: ≥ 80% (blastocysts after 120h in standard mouse IVF medium covered with test item)
- Chemical composition
- Use of Pharmaceutical grade mineral (paraffin) oil
- Certificate of Analysis and MSDS are available upon request

### Sterility

STERILE A

### Precautions and Warnings

Use aseptic techniques to avoid possible contamination. Always wear protective clothing when working with specimens. Handle all human, organic material as if capable of transmitting HIV or hepatitis. Handle all human, organic material as if potentially infectious.

When the oil is removed from the incubator and placed at room temperature for an extended period of time the oil may appear cloudy. Once re-equilibrated, by placing the oil back in the incubator for multiple hours, the cloudiness usually disappears. In case the cloudiness does not disappear discard the product.

Any Serious Incident (as defined in European Medical Device Regulation 2017/745) that has occurred should be reported to Gynotec B.V. and, if applicable, the competent authority of the EU Member State in which the user and/or patient is established.

### Pre-use checks

- Do not use the product if:
  - the seal on the bottle is broken or open when the product is delivered;

- it becomes cloudy or the product shows any signs of microbial contamination;
- expiry date has been exceeded.
- Do not freeze before use.
- Do not re-sterilize after opening.
- Keep product in original packaging when not in use.

### Storage Conditions

Store at 15°-25°C.

(Storage at 2°-8°C is allowed, but can cause turbidity which will disappear once the oil is warmed up again.)

Keep away from (sun)light.

Do not expose to sources of irradiation.

After opening the container, the product can be used up to 28 days.

Sterile conditions must be maintained and product must be stored at room temperature, or alternatively 2°-8°C.

Stable after transport (up to 5 days) at elevated temperatures (≤ 37°C).

Discard the devices in accordance with local regulations for disposal of medical devices.

### Clinical Benefit

The clinical benefit of IVF Basics® Cult-Oil is related to the fact that the oil is used during ART-procedures of patients and couples with infertility problems. It is expected that the outcomes of ART procedures in which the oil is used are comparable with the ART outcomes (i.e. pregnancy rate and delivery rate) as published in the annual peer-reviewed report of the European IVF-monitoring Consortium (EIM) for the European Society of Human Reproduction and Embryology (ESHRE).

For further questions regarding to the safety and performance, please contact Gynotec B.V. for technical or customer support.

## Instructions for use (Suggested procedures)

IVF Basics® Cult-Oil is pre-washed. No further preparations necessary.

IVF Basics® Cult-Oil can be used together with a handling or embryo/gamete culture media classified as a medical device class IIb or III such as e.g. IVF Basics® HTF (Gynotec B.V.).

### Instructions

- Preferably prepare the dishes/bottles with mineral oil one day before use.
- Incubate the dishes/bottles at 37°C in a CO<sub>2</sub> incubator. This way the mineral oil achieves the preferred warmth and becomes saturated with gas.
- In case bottles are used, do not close the cap on the bottle in order to allow gas exchange.
- Important notice: For further instructions, please refer to the validated procedures and protocols established in your laboratory.

SYMBOLS GLOSSARY			
	<b>Catalogue number</b>		Batch code
	Use-by date		<b>Manufacturer</b>
	<b>Keep away from sunlight</b>		Medical Device
	Consult instructions for use		<b>Cautions</b>
	<b>Date of manufacture</b>		Sterilized using aseptic processing techniques
	Temperature limit		<b>Single sterile barrier system</b>
	<b>Do not use if package is damaged and consult instructions for use</b>		Single sterile barrier system with protective packaging outside
Symbol as defined in EU Commission MDD 93/42/EEC			
	CE marking by Notified Body 0344		
Symbol as defined in the Resolution of the Cabinet of Ministers of Ukraine No. 1184 dated 30.12.2015			
	Mark of conformity with technical regulations accompanied by the identification number of the designated conformity assessment body		

**Technical – Customer Support**

**Gynotec B.V.**  
 Rijksweg 150  
 6581 ET Malden  
 The Netherlands  
 Tel.: +31 (0)24 3586582  
 Fax: +31 (0)24 3581355  
 URL: [www.gynotec.com](http://www.gynotec.com)  
 Email: [info@gynotec.nl](mailto:info@gynotec.nl)

**Ukrainian Authorized Representative**

Stoik Company LLC  
 C 04136 ap. A-612, Building A, 13 Ivana Vygovskogo str., Kyiv,  
 Ukraine  
 Tel.: +38 (0)67 112-92-92  
 URL: [www.stoik.com.ua](http://www.stoik.com.ua)  
 Email: [stoik.ivf@gmail.com](mailto:stoik.ivf@gmail.com)

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